



# ALLERGY MATRIX

Advanced. Simple. Comprehensive.

Allergy Testing the way you need it.

- ▶ 5 Drops Of Blood
- ▶ 44 Environmental Allergens
- ▶ 60 Food Allergens

COMPLIANT, TRANSPARENT AND  
COMPASSIONATE BILLING MEANS  
NO PATIENT SURPRISES.



- ▶ Results integrated to immunotherapy portal
- ▶ All commercial and federal insurances accepted
- ▶ CLIA and COLA certified lab

Dust Mite				
CLASS	let			
0	0.00			
0	0.00			
0	0.00			
Molds I				
CLASS	let	#W	BLOCKING POT	
0	0.00	0.40	0.00	
0	0.00	0.00	0.00	
0	0.00	0.00	0.00	
Molds II				
CLASS	let	#W		
0	0.00	0.44		
0	0.00	0.00		
Dander				
CLASS	let	#W		
0	0.00	0.00		
0	0.00	0.00		
0	0.00	0.00		
Weeds				
CLASS	let	#W		
0	0.00	2.15		
0	0.00	0.00		
0	0.00	0.00		
0	0.00	0.00		
0	0.00	0.00		
0	0.00	0.00		
0	0.00	0.00		
0	0.00	0.00		
0	0.00	0.00		
0	0.00	0.00		

7

[www.GreystoneDiagnostics.com](http://www.GreystoneDiagnostics.com)



# EXPEDITED SHIPPING INSTRUCTIONS

When you need results back quickly, use the provided UPS 2<sup>nd</sup> Day Air Express Mailers.

01

Flip past first 2 pages of provided label to reveal sticky label



02

Affix label to front of mailer envelope



03


Place demographics, completed req, and sample in envelope



Please email us at [support@greystonediagnostics.com](mailto:support@greystonediagnostics.com)  
if you have any questions.

# DRIED BLOOD SPOT SPECIMEN COLLECTION INSTRUCTIONS

- 01.** Clearly label the collection card.
- 02.** Disinfect fingertip with alcohol pad.
- 03.** Prick fingertip using lancet/needle.
- 04.** Allow full drop of blood to collect on finger. Use cotton to wipe this drop.
- 05.** Collect the remaining drops of blood onto collection card. Collect 5 full drops of blood. (Drop of blood should fully form before dropping on to filter paper).
- 06.** Hold cotton on fingertip until bleeding stops.
- 07.** Allow filter paper to dry before mailing

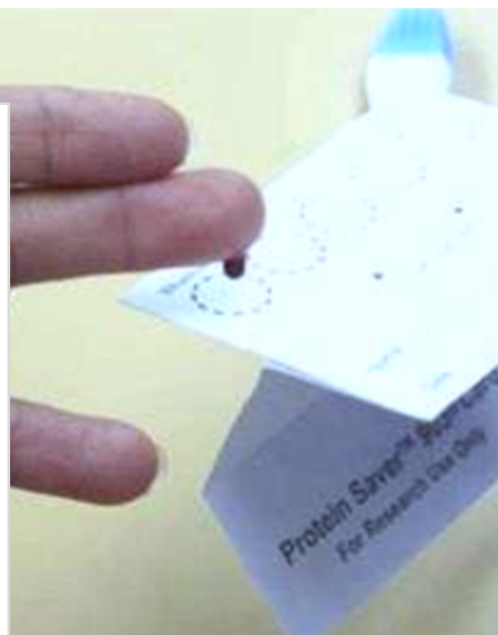

See example of correctly filled  
collection card here

